BTE

Alta2 Pro Ti
Nera2 Pro Ti
Ria2 Pro Ti

Instructions for use

oticon
PEOPLE FIRST

Model overview

This booklet is valid for Oticon BTE in the following hearing aid families:

- Alta2 Pro Ti
- Nera2 Pro Ti
- Ria2 Pro Ti

BTE is available in fitting level 85.

Introduction to this booklet

This booklet guides you in how to use and maintain your new hearing instrument. Please read the booklet carefully including the **Warnings section**. This will help you to achieve the full benefit of your new hearing instrument.

Your Hearing Care Professional has adjusted the hearing instrument to meet your needs. If you have additional questions, please contact your Hearing Care Professional.

| **About** | [Start up](#) | [Handling](#) | [Options](#) | [Tinnitus](#) | [Warnings](#) | [More Info](#) |

For your convenience this booklet contains a navigation bar to help you navigate easily through the different sections.

Indication for use

The hearing aid is intended to amplify and transmit sound to the ear and thereby compensate for impaired hearing within mild to severe-to-profound hearing loss. The hearing aid is not intended to be used by infants/children < 36 months.

IMPORTANT NOTICE

The hearing instrument amplification is uniquely adjusted and optimized to your personal hearing capabilities during the instrument fitting performed by your Hearing Care Professional.

Table of contents 1/2

About			
	Identify your hearing instrument style and ear piece	10	
	BTE with sound hook and mold	12	
	BTE with thin tube (Corda miniFit)	14	
	Identify left and right instrument	16	
	Battery	17	
Start up			
	Turn the hearing instrument on and off	18	
	When to replace a battery	19	
	Replace the battery	20	
Handling			
	Caring for your hearing instrument	22	
	MultiTool for handling batteries and cleaning	23	
	Sound hook with mold	24	
	Put on the instrument with mold	25	
	Clean the mold	26	
	Wash the mold	27	
	Thin tube (Corda miniFit)	28	
	Put on the instrument with thin tube	29	
	Clean the thin tube	30	
	Standard earpieces: dome or Grip Tip	32	
	Replace dome or Grip Tip	33	
	Thin tube with Micromold or LiteTip	34	
	Maintain the Micromold or LiteTip	35	
	Replace ProWax filter	36	
Options			
	Optional features and accessories	38	
	Mute the hearing instrument	39	
	Change programs	40	
	Change volume	42	
	Tamper-resistant battery drawer	44	
	Wireless accessories	46	
	Other options	48	

Continues on next page

Table of contents 2/2

Tinnitus

Tinnitus SoundSupport™ (optional)	50
Guidelines for tinnitus sound generator users	51
Sound options and volume adjustment	52
Limitation on use time	55
Important information for hearing care professionals about Tinnitus SoundSupport	56
Warnings related to Tinnitus	59

Warnings

Warnings	60
----------	----

More Info


Troubleshooting guide	66
Water resistance	68
Conditions of use	69
Warranty certificate	70
International warranty	71
Technical information	72

Identify your hearing instrument style and ear piece


There are two different attachments and several different ear pieces which classify your hearing instrument style. Please identify your attachment and ear piece. This will make it easier for you to navigate through this booklet.

BTE with sound hook and mold

An instrument with a sound hook will have a mold as an ear piece.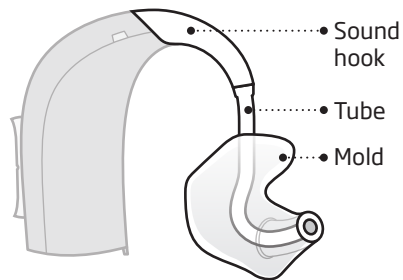


BTE with thin tube (Corda miniFit)

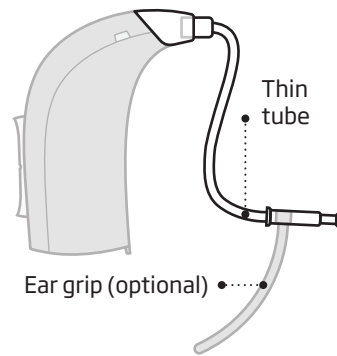Earpieces


An instrument with a thin tube will have one of the following ear pieces - dome, GripTip, micro mold, or LiteTip.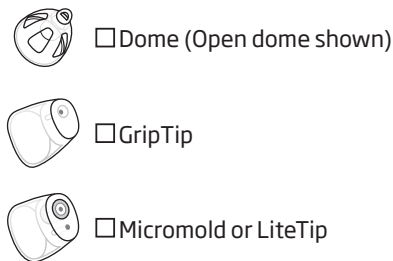


* Please see details for replacing the dome or Grip Tip in section: Replace dome or Grip Tip

BTE with sound hook and mold

What it is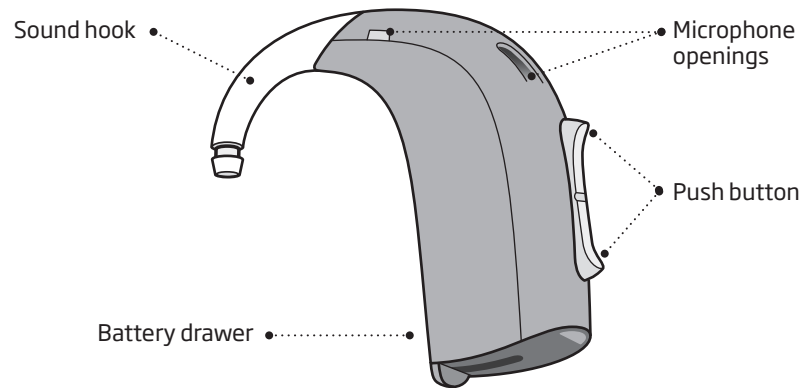


What it does

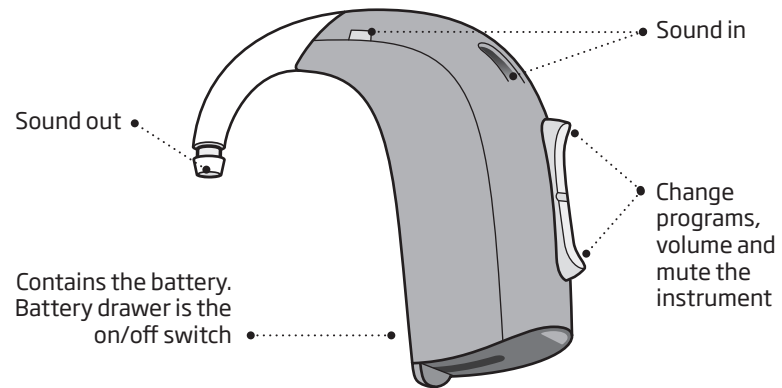
BTE with thin tube (Corda miniFit)

What it is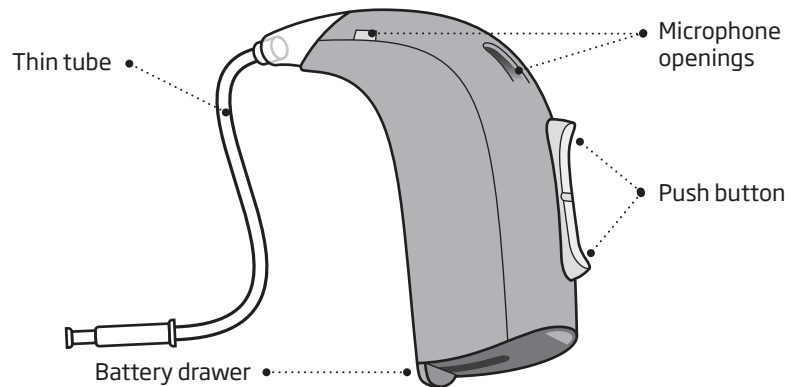


What it does

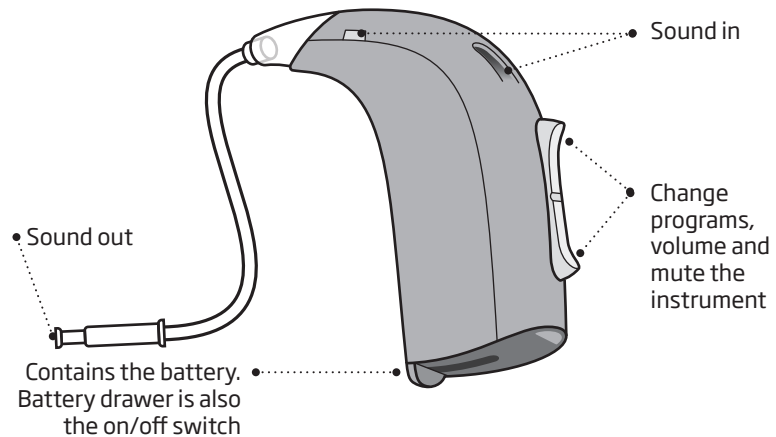
Identify left and right instrument

It is important to distinguish between the left and the right instrument as they might be programmed differently.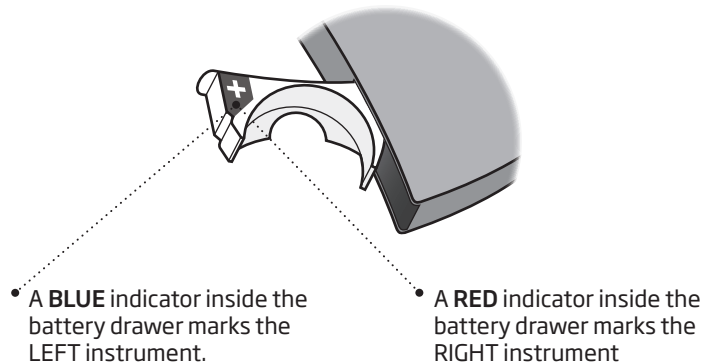

Battery


Your hearing instrument is a miniature electronic device that runs on special batteries. To activate the hearing instrument, you must insert a new battery in the battery drawer. See how in the “Replace the battery” section.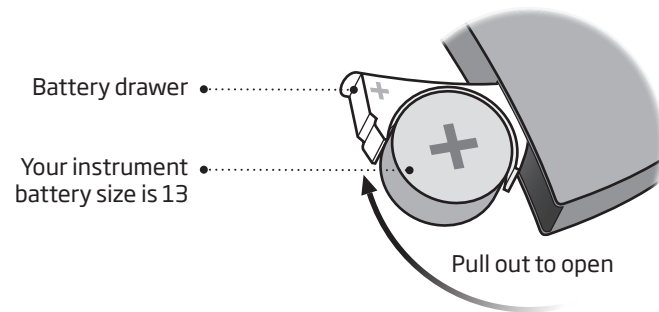


Turn the hearing instrument on and off

The battery drawer is also used to switch the hearing instrument on and off. To preserve the battery, make sure your instrument is switched off when you are not wearing it.

Turn ON
Close the battery drawer with the battery in place.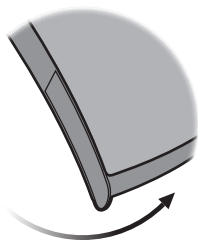

Turn OFF
Open the battery drawer

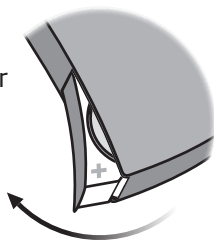When to replace a battery

When it is time to replace the battery you will hear two beeps repeated in moderate intervals until the battery runs out.

Two beeps
= The battery is running low


Four beeps
= The battery has run out

Battery maintenance tip

To make sure the hearing instrument is always working bring spare batteries with you, or replace the battery before you leave home.


Replace the battery

1. Remove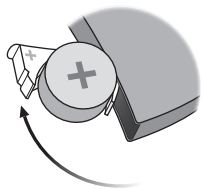


Fully open the battery drawer. Remove the battery.

2. Uncover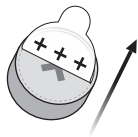


Remove the sticky label from the + side of the new battery.

3. Insert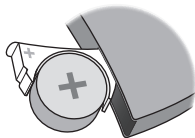


Insert the new battery into the battery drawer. Make sure the + side faces up.

4. Close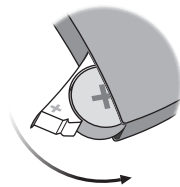

Close the battery drawer. The instrument will play a jingle through the ear piece. Hold the earpiece close to your ear to hear the jingle.

Tip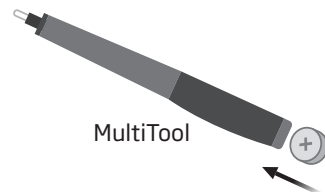

The MultiTool can be used for battery change. Use the magnetic end to remove and insert batteries.


The MultiTool is provided by your Hearing Care Professional.

Caring for your hearing instrument

When handling your hearing instrument, hold it over a soft surface to avoid damage if you drop it.

Clean the microphone openings

Carefully brush away debris from the openings. Gently brush the surface. Make sure the brush is not pressed in to the openings.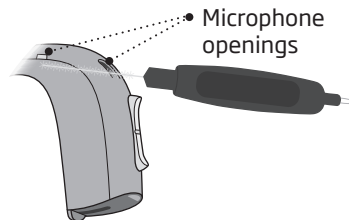


IMPORTANT NOTICE

Use a soft, dry cloth to clean the hearing instrument. It must never be washed or immersed in water or other liquids

MultiTool for handling batteries and cleaning

The MultiTool contains a magnet which makes it easier to replace the battery in the hearing aid. It also contains a brush and a wire loop for cleaning and removing ear wax. If you need a new MultiTool please contact your hearing care professional.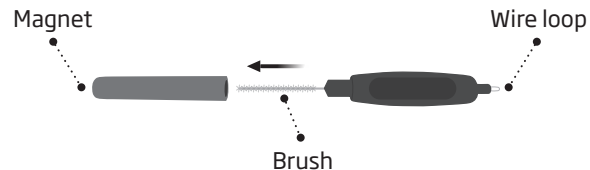


IMPORTANT NOTICE

The MultiTool has a built-in magnet. Keep the MultiTool 30 cm away from credit cards and other magnetically sensitive devices.

Sound hook with mold

Read the following pages if you have a hearing instrument with a sound hook and a mold.

The mold is customized for you and fits your ear.

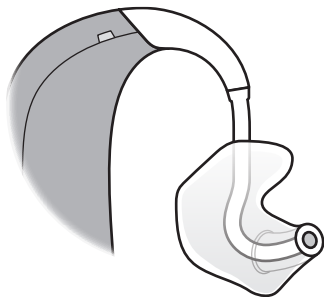Put on the instrument with mold

Step 1

Gently pull your ear outwards and press the mold in the direction of the ear canal, twisting slightly.


Step 2

Place the hearing instrument behind your ear.

Clean the mold

The mold should be cleaned regularly. Use a soft cloth to clean the surface of the mold. Use the MultiTool wire loop to clean the opening.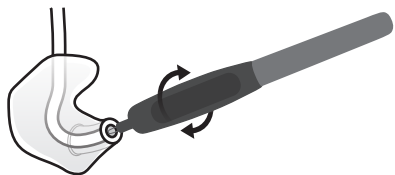

Replace the tube

The tubing between the mold and the instrument should be replaced when it becomes yellow or stiff. Consult your Hearing Care Professional in relation to this.

Wash the mold

Step 1


Make sure you are able to identify which mold is for which instrument before disconnecting them from the instruments.

Step 2

Disconnect the tube and mold from the hook. Hold the hook firmly when pulling off the tube. Otherwise the instrument may be damaged.

Step 3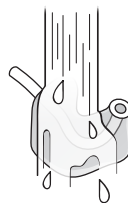


Wash the mold and tube using mild soap. Rinse and dry completely before reconnecting to the hearing instrument.

Thin tube (Corda miniFit)

Read the following pages if your instrument has a thin tube (Corda miniFit) with either a dome, GripTip, Micromold or LiteTip ear piece.

Corda miniFit is a thin tube that brings the sound into your ear. The thin tube should always be used with an ear piece attached. Use only parts designed for your hearing instrument.

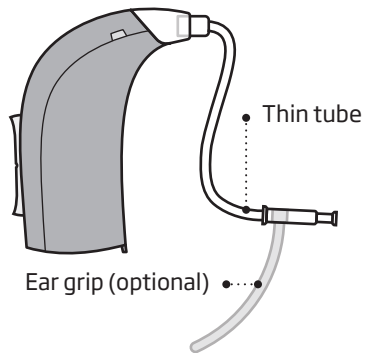
Put on the instrument with thin tube

Step 1


Place the hearing instrument behind your ear.

Step 2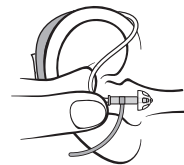

Hold the bend of the tube between your thumb and index finger. The ear piece should point towards the ear canal opening.

Step 3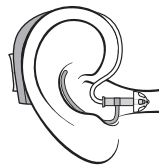


Gently push the ear piece into your ear canal until the thin tube sits close against the side of your head.

Clean the thin tube


The thin tube should be cleaned to keep it clear from moisture and wax. If not cleaned it could become blocked with wax which means no sound will be able to come out.

Step 1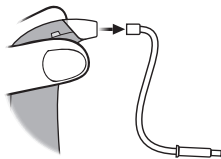


Pull the thin tube out of the hearing instrument

Step 2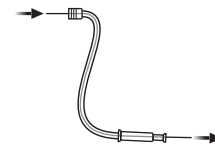

Push the cleaning tool all the way through the thin tube.


Step 3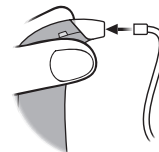

Reconnect the thin tube to the hearing instrument.

Standard earpieces: dome or Grip Tip

Both domes and Grip Tip are made of soft rubber material. There are 4 different types of domes. Check your dome type and size below.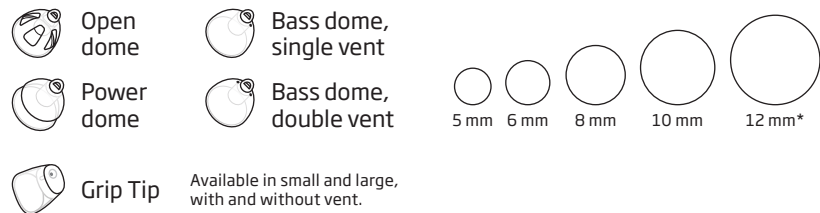

* not for open dome


IMPORTANT NOTICE

If the earpiece is not on the speaker when removed from the ear, the earpiece might still be in the ear canal. Consult your hearing care professional for further instructions.

Replace dome or Grip Tip


The standard earpiece (dome or Grip Tip) should not be cleaned. If the earpiece is filled with wax, replace it with a new one (typical weekly). Grip Tips should be replaced at least once a month.

Step 1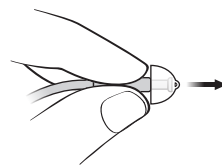


Hold on to the end of the thin tube and then pull off the dome.

Step 2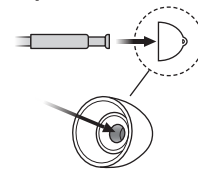

Insert the thin tube exactly into the middle of the earpiece to obtain a secure attachment.

Step 3

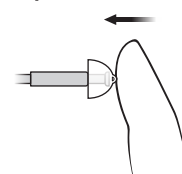Push firmly to ensure that the dome is fastened securely.

Thin tube with Micromold or LiteTip

Read the following pages if you have a Micromold or LiteTip ear piece. The Micromold and LiteTip are customized for your ear shape.

• Micromold


• LiteTip

Maintain the Micromold or LiteTip

The ear piece should be cleaned regularly.

The wax filter in the ear piece should be replaced when clogged or when the hearing instrument does not sound normal.

Use the wax filter delivered by your Hearing Care Professional.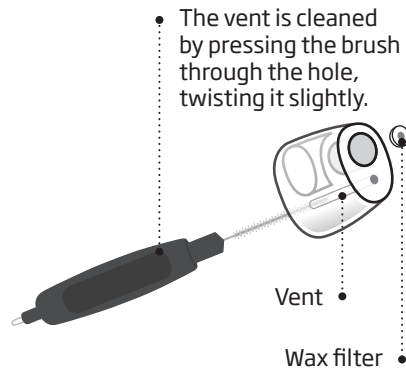


IMPORTANT NOTICE

Always use the same type of wax filter as was originally supplied with the hearing aid. If you are in any doubt about the use or replacement of wax filters, contact your hearing care professional.


Replace ProWax filter

1. Tool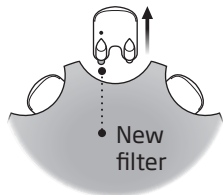


Remove the tool from the shell. The tool has two pins, one empty for removal and one with the new wax filter.

2. Remove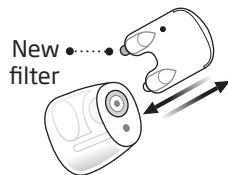

Push the empty pin into the wax filter in the Micromold and pull it out.

3. Insert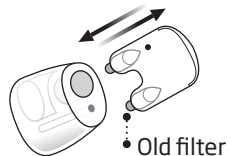

Insert the new wax filter using the other pin, remove the tool, and throw it out.

Optional features and accessories


The features and accessories described on the following pages are optional. Please contact your Hearing Care Professional to find out how your hearing instrument is programmed.

If you experience difficult listening situations, a special program may be helpful. These are programmed by your Hearing Care Professional.

Write down hearing situations where you may need help.

Mute the hearing instrument

Use the mute function if you need to silence the instrument while wearing it.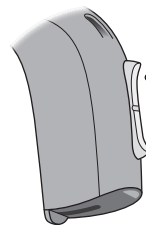


Apply a very long press to either end of the button to mute the instrument. To reactivate the instrument, push the button briefly.

IMPORTANT NOTICE

Do not use the mute function as an off switch, as the hearing instrument still draws current from the battery in this mode.

Change programs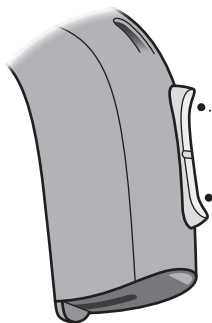

Your hearing instrument can have up to 4 different programs. These are programmed by your Hearing Care Professional.

Press up or down
between programs

Note that you can change continuously between programs - both up and down in the program order. If for example you want to go from program 1 to 4, you can press the down button once instead of pressing the up button 3 times.

To be filled out by the Hearing Care Professional

Program	Sound you will hear when activated	When to use
1	 "1 beep"	
2	 "2 beeps"	
3	 "3 beeps"	
4	 "4 beeps"	

Program change:

LEFT


RIGHT

Short press

Long press

Change volume

The push button allows you to adjust the volume.
You will hear a click when you turn up or down the volume.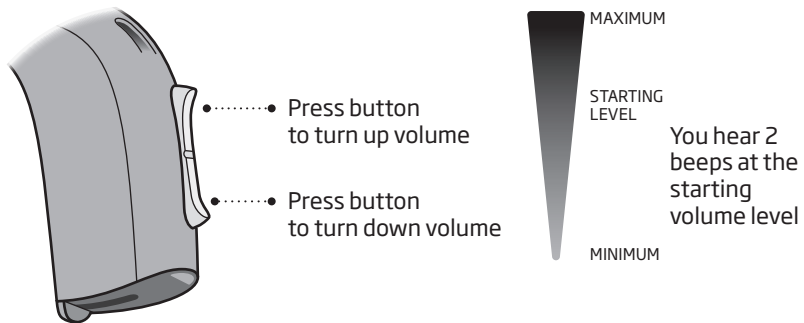


To be filled out by the Hearing Care Professional

Volume change	<input type="checkbox"/> LEFT	<input type="checkbox"/> RIGHT	<input type="checkbox"/> Short press
---------------	-------------------------------	--------------------------------	--------------------------------------


Quick reset

If you wish to return to the standard settings of the instrument programmed by your Hearing Care Professional, simply open and then close the battery drawer.

Open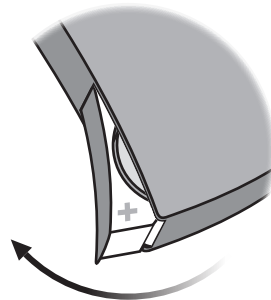

Close

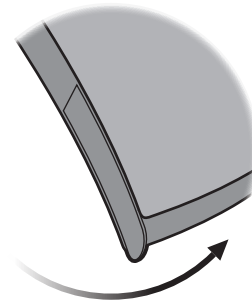□ Tamper-resistant battery drawer


To keep the battery out of reach of infants, small children and people with learning difficulties, a tamper-resistant battery drawer should be used. Use a small screwdriver to lock and unlock the drawer.

IMPORTANT NOTICE

Avoid excessive force to open the battery drawer when in the locked position. Do not force the battery drawer beyond its fully opened position. Make sure to insert the battery correctly.


Please contact your Hearing Care Professional if you suspect the tamper-resistant effectiveness and locking ability is damaged.

Unlock the battery drawer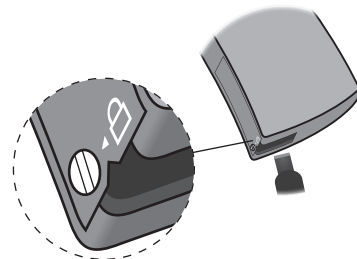

Use the screwdriver to turn the screw to unlock position (vertical). Insert the screwdriver into the slit in the bottom of the hearing instrument and push the door open.

Lock the battery drawer

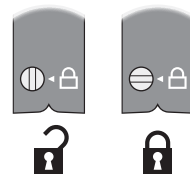Make sure the drawer is completely closed. Turn the screw to the locked position (horizontal).

Wireless accessories

As an enhancement to your wireless hearing instrument a broad range of wireless accessories are available.

ConnectLine

ConnectLine is a family of products that allows you to receive audio signals from TVs, phones, music players, PCs or an external microphone wirelessly through your hearing instrument.

Remote Control

The Remote Control offers an opportunity to change program or adjust the volume in your hearing instrument.

FM

Your hearing instrument supports the option of attaching an FM system. The FM system allows you to receive speech or audio signals directly to your hearing instrument from a dedicated transmitter.

For detailed information on wireless accessories contact your Hearing Care Professional or visit www.oticon.ca

Other options

Telecoil

helps you hear better when using a telephone with a built in loop or when you are in buildings with teleloop systems such as theaters, churches or lecture rooms. This symbol or a similar sign is shown wherever a tele loop has been installed.

Autophone

is a program that can be automatically activated if your telephone has a built in tele loop (see above). A magnet may need to be placed on your telephone next to the sound outlet.

Direct Audio Input (DAI)

allows your hearing instrument to receive signals directly from external sources such as TV, radio, music players, etc. The DAI adaptor is mounted to your instrument and via a cable connected to an external sound source.

For more information on these options contact your Hearing Care Professional or visit www.oticon.ca.

IMPORTANT NOTICE

When the DAI is connected to an audio source plugged into a wall outlet, the audio source must comply with IEC-60065, IEC-60601 or equivalent safety standards.

Tinnitus SoundSupport™ (optional)

Intended use of Tinnitus SoundSupport

Tinnitus SoundSupport is a tool intended to generate sounds to provide temporary relief for patients suffering from tinnitus as part of a tinnitus management program.

The target population is the adult population (over 18 years old).

Tinnitus SoundSupport is targeted to licensed hearing care professionals (audiologists, hearing aid specialists, or otolaryngologists) who are familiar with the evaluation and treatment of tinnitus and hearing loss. Fitting of Tinnitus SoundSupport must be done by a hearing care professional participating in a tinnitus management program.

Guidelines for tinnitus sound generator users

These instructions contain information about Tinnitus SoundSupport, which may have been enabled in your hearing aids by your hearing care professional.

Tinnitus SoundSupport is a tinnitus management device intended to generate sound of sufficient intensity and bandwidth to help manage tinnitus.

Your hearing care professional will also be able to offer the appropriate follow-up care. It is important to follow his/her advice and directions regarding such care.

Prescription use only

Good health practice requires that a person reporting tinnitus have a medical evaluation by a licensed ear physician before using a sound generator. The purpose of such an evaluation is to ensure that any medically treatable condition that may cause tinnitus is identified and treated prior to using a sound generator.

Sound options and volume adjustment

Tinnitus SoundSupport is programmed by your hearing care professional to match your hearing loss and preferences for tinnitus relief. It offers a number of different sound options. Together with your hearing care professional, you can select your preferred sound(s).

Tinnitus SoundSupport programs

Together with your hearing care professional you decide for which programs you may want to have Tinnitus SoundSupport activated. The sound generator can be activated in up to four different programs.

Mute

If you are in a program for which Tinnitus SoundSupport is activated, the mute functionality will mute only the environmental sounds, and not the sound from Tinnitus SoundSupport.

See chapter: "Mute the hearing aid".

Volume adjustments with Tinnitus SoundSupport

When you select a hearing aid program for which Tinnitus SoundSupport is activated, your hearing care professional can only set the push button on your hearing aid to work as a volume control for the tinnitus relief sound.

Your hearing care professional will set the volume control for the sound generator in one of two ways:

- A) Change volume in each ear separately, or
- B) Change volume in both ears simultaneously.

<input type="checkbox"/>	<p>A) How to change Tinnitus SoundSupport volume in each ear separately To increase volume (on one hearing aid only), use a short press on the upper part of the push button repeatedly until desired level is reached. To decrease volume (on one hearing aid only), use a short press on the lower part of the push button repeatedly until desired level is reached.</p>
<input type="checkbox"/>	<p>B) How to change Tinnitus SoundSupport volume in both ears simultaneously You can use one hearing aid to increase/decrease the sound in both hearing aids. When changing the volume in one hearing aid, the volume on the other hearing aid will follow. To increase volume, use a short press on the upper part of the push button repeatedly. To decrease volume, use a short press on the lower part of the push button repeatedly.</p>

To be filled out by your hearing care professional.

Limitation on use time

Daily use

The volume levels of Tinnitus SoundSupport can be set to a level which could lead to permanent hearing damage when used for a prolonged period of time. Your hearing care professional will advise you of the maximum amount of time per day you should use Tinnitus SoundSupport. It should never be used at uncomfortable levels.

Tinnitus SoundSupport: Limitation on use

<input type="checkbox"/>	No limitation on use
<input type="checkbox"/>	Limited use, maximum _____ hours per day

To be filled out by your hearing care professional.

Important information for hearing care professionals about Tinnitus SoundSupport

Device description

Tinnitus SoundSupport is a module function that can be enabled in the hearing aids by the hearing care professional.

Maximum wearing time

The wearing time of Tinnitus SoundSupport will decrease as you increase the level above 80 dB(A) SPL. The fitting software will automatically display a warning when the hearing aid exceeds 80 dB(A) SPL. See Wearing Time Indicator next to the tinnitus fitting graph in the fitting software.

If the volume control is enabled

By default the volume control for the sound generator is deactivated in the instrument. Risk of noise exposure increases when the volume control is enabled.

In the Tinnitus volume control tab on the “Buttons & Indicators” screen a warning will be displayed when the highest volume control setting in the instrument exceed 80 dB(A) SPL.

Using the table provided, calculate the maximum wearing time for your patient based on:

1. The output of the program with the highest dB(A)SPL level.*
plus
2. the decibel range of the volume control.

** Can be found by hovering over signal level control.*

Instruct your patient accordingly and record your patient's wearing time in the table "Tinnitus SoundSupport: Limitation on use", in section "Limitation on use time".

Maximum wearing time of Tinnitus SoundSupport

total dB(A) SPL	Wearing time/hours per day
< 81	Full day
82	16
85	8
88	4
90	3

Ref. OSHA (Occupational Safety & Health Administration, U.S. department of Labour) / DIRECTIVE 2003/10/EC on the minimum health and safety requirements regarding the exposure of workers to the risks arising from physical agents (noise) / Occupational exposure to noise: evaluation, prevention and control - Special Report S 64, WHO.

Warnings related to Tinnitus

If your hearing care professional has activated the sound generator Tinnitus SoundSupport, please pay attention to the following warnings.

There are some potential concerns associated with the use of any sound generated by a tinnitus management device. Among them are the potential worsening of tinnitus, and/or a possible change in hearing thresholds.

Should you experience or notice a change in hearing or tinnitus, or any dizziness, nausea, headaches, heart palpitations, or possible skin irritation at the point of contact with the device, you should immediately discontinue use of the device and consult a medical, audiology, or other hearing care professional.

As with any device, misuse of the sound generator feature may cause potentially harmful effects. Care should be taken to prevent unauthorized use and to keep the device out of reach of children and pets.

Maximum wearing time

Always follow the maximum wearing time per day of the Tinnitus SoundSupport advised by your hearing care professional. Prolonged use may lead to worsening of your tinnitus or of your hearing loss.

Warnings

You should familiarize yourself fully with the following general warnings before using your hearing aid for your personal safety and to ensure correct use.

Please note that a hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Furthermore, note that in most cases, infrequent use of a hearing aid does not permit a user to attain full benefit from it.

Consult your hearing care professional if you experience unexpected operations or events with your hearing aid.

Usage of hearing aids

- Hearing aids should be used only as directed and adjusted by your hearing care professional. Misuse can result in sudden and permanent hearing loss.
- Never allow others to wear your hearing aid as incorrect usage could cause permanent damage to their hearing.

Choking hazards & risk of swallowing batteries

- Hearing aids, their parts, and batteries should be kept out of reach of children and anyone who might swallow these items, or otherwise cause injury to themselves.

- Batteries have occasionally been mistaken for pills. Therefore, check your medicine carefully before swallowing any pills.
- Most hearing aids can be supplied with a tamper-resistant battery drawer upon request. This is strongly recommended for infants, small children and people with learning difficulties.
- Children younger than 36 months must always use a tamper-resistant battery drawer. Please talk to your hearing care professional about the availability of this option.

If a battery or hearing aid is swallowed, see a doctor immediately.

Battery use

- Always use batteries recommended by your hearing care professional. Batteries of low quality may leak and cause bodily harm.
- Never attempt to recharge your batteries and never dispose of batteries by burning them. There is a risk that the batteries will explode.

Warnings

Dysfunction

- Be aware of the possibility that your hearing aid may stop working without notice. Keep this in mind when you depend on warning sounds (e.g. when you are in traffic). The hearing aids may stop functioning, for instance if the batteries have expired or if the tubing is blocked by moisture or ear wax.

Active implants

- Caution must be taken with active implants. In general, follow the guidelines recommended by manufacturers of implantable defibrillators and pacemakers regarding use with mobile phones and magnets.
- The Autophone magnet and MultiTool (which has a built-in magnet) should be kept more than 30 cm away from the implant, e.g. do not carry it in a breast pocket.

- If you have an active brain implant, please contact the manufacturer of your implantable device for information about the risk of disturbance.

X-ray, CT, MR, PET scanning and electrotherapy

- Remove your hearing aid before X-ray, CT/MR/PET scanning electrotherapy, surgery etc. as your hearing aid may be damaged when exposed to strong fields.

Heat and chemicals

- The hearing aid must never be exposed to extreme heat e.g. left inside a parked car in the sun.
- The hearing aid must not be dried in microwave ovens or other ovens.
- The chemicals in cosmetics, hairspray, perfume, aftershave lotion, suntan lotion and insect repellent can damage the hearing aid. Always remove your hearing aid before applying such products and allow time to dry before use.

Warnings

Possible side effects

- Hearing aids, molds or domes may cause an accelerated accumulation of ear wax.
- The otherwise non-allergenic materials used in hearing aids may in rare cases cause a skin irritation or other side effects.

Please seek consultation with a physician if these conditions occur.

Interference

- The hearing aid has been thoroughly tested for interference, according to the most stringent international standards. However, interference between the hearing aid and other devices (e.g. some mobile telephones, citizens band systems and shop alarm systems and other devices) may occur. If this occurs, increase the distance between the hearing aid and the interfering device.

Connection to external equipment

- The safety of the hearing aid when connected to external equipment (via auxiliary input cable, via USB cable, or directly), is determined by the external equipment. When connected to external equipment plugged into a wall outlet, this equipment must comply with IEC-60065, IEC-60950 or equivalent safety standards.

Troubleshooting guide

Symptom	Possible causes	Solutions
No sound	Worn-out battery	Replace the battery
	Clogged tube or earpieces (mold, dome, GripTip, Micromold or LiteTip)	Clean mold Clean Corda miniFit (thin tube), replace dome or GripTip or replace wax filter
Intermittent or reduced sound	Clogged sound outlet	Clean mold Clean Corda miniFit (thin tube), replace dome or GripTip or replace wax filter
	Moisture	Wipe battery with a dry cloth and/or dry tube
	Worn-out battery	Replace the battery
	Clogged damping filter in sound hook	Contact your Hearing Care Professional for assistance
Squealing noise	Hearing instrument earpiece inserted incorrectly	Re-insert earpiece (mold, dome, GripTip, Micromold or LiteTip)
	Ear wax accumulated in ear canal	Have ear canal examined by your doctor

If none of the above solutions work, consult your Hearing Care Professional for assistance.

Water resistance

Your hearing instrument is water resistant which means it is designed to be worn in all daily life situations. Therefore you do not have to worry about sweat or getting wet in the rain. Should your hearing instrument come in contact with water and stop working, please follow these guidelines:

1. Gently wipe off any water
2. Open the battery drawer and remove the battery and gently wipe off any water in the battery drawer
3. Let the hearing instrument dry with the battery drawer left open for approximately 30 minutes
4. Insert a new battery

IMPORTANT NOTICE

Do not wear your hearing instrument while showering or participating in water activities. Do not immerse your hearing instrument in water or other liquids.

Conditions of use

Operating conditions	Temperature: +1°C to +40°C Relative humidity: 5% to 93%, non-condensing
Storage and transportation conditions	Temperature and humidity should not exceed the following limits for extended periods during transportation and storage: Temperature: -25°C to +60°C Relative humidity: 5% to 93%, non-condensing

Warranty certificate

Name of owner: _____

Hearing care professional: _____

Hearing care professional address: _____

Hearing care professional phone: _____

Purchase date: _____

Warranty period: _____ Month: _____

Model left: _____ Serial no.: _____

Model right: _____ Serial no.: _____

International warranty

Your hearing instrument is covered by an international limited warranty issued by the manufacturer for a period of 12 months from the date of delivery. This limited warranty covers manufacturing and material defects in the hearing instrument itself, but not accessories such as batteries, tubing, ear wax filters etc. Problems arising from improper handling or care, excessive use, accidents, repairs made by an unauthorized party, exposure to corrosive conditions, physical changes in your ear, damage due to foreign objects entering the device, or incorrect adjustments are NOT covered by the limited warranty and may void it. The above warranty does not affect any legal rights that you

might have under applicable national legislation governing sale of consumer goods. Your Hearing Care Professional may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult him/her for further information.

If you need service

Take your hearing instrument to your Hearing Care Professional, who may be able to sort out minor problems and adjustments immediately.

Technical information

The hearing instrument contains a radio transmitter using short range magnetic induction technology working at 3.84 MHz. The magnetic field strength of the transmitter is < -42 dB μ A/m @ 10m.

The hearing aid complies with international standards concerning radio transmitters, electro-magnetic compatibility and human exposure.

Due to the limited space available on the instruments all relevant approval markings are found in this document.

USA and Canada

The hearing aid contains a radio module with the following certification ID numbers:

FCC ID: U28FU2BTERIT
IC: 1350B-FU2BTERIT

The device complies with Part 15 of the FCC Rules and with Industry Canada's licence-exempt RSSs.

Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

More information: www.oticon.ca

This Class B digital apparatus complies with Canadian ICES-003.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined

by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the manufacturer or an experienced radio/TV technician for help.

Oticon declares that this hearing instrument is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/UE (RED).

Declaration of Conformity is available at the manufacturer.

Manufactured by:

Oticon A/S
Kongebakken 9
DK-2765 Smørum
Denmark
www.oticon.global

CE 0543

Waste from electronic equipment must be handled according to local regulations.

Imported and Distributed by:

Audmet Canada Ltd
1600-4950 Yonge St
Toronto, ON M2N 6K1
www.oticon.ca

Settings overview for your hearing instrument				
Left			Right	
<input type="checkbox"/> Yes	<input type="checkbox"/> No	Volume control	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<input type="checkbox"/> Yes	<input type="checkbox"/> No	Program shift	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<input type="checkbox"/> Yes	<input type="checkbox"/> No	Mute	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<input type="checkbox"/> Yes	<input type="checkbox"/> No	Tinnitus SoundSupport	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Volume control indicators				
<input type="checkbox"/> On	<input type="checkbox"/> Off	Beeps at min /max volume	<input type="checkbox"/> On	<input type="checkbox"/> Off
<input type="checkbox"/> On	<input type="checkbox"/> Off	Clicks when changing volume	<input type="checkbox"/> On	<input type="checkbox"/> Off
<input type="checkbox"/> On	<input type="checkbox"/> Off	Beeps at preferred volume	<input type="checkbox"/> On	<input type="checkbox"/> Off
Battery indicators				
<input type="checkbox"/> On	<input type="checkbox"/> Off	Low battery warning	<input type="checkbox"/> On	<input type="checkbox"/> Off

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